




How to use


Measure

82 -140cm = Small /Medium
120-215cm = Large /X-Large

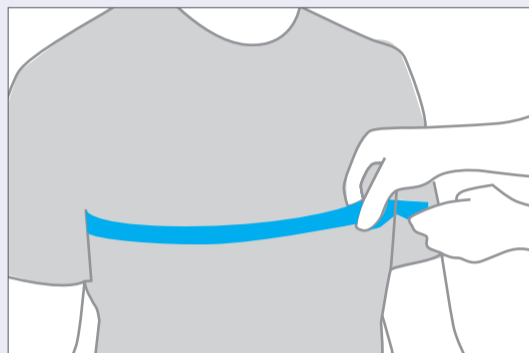


Ensure that the correct size is given by measuring underneath the armpits.

For Coughing... Pull

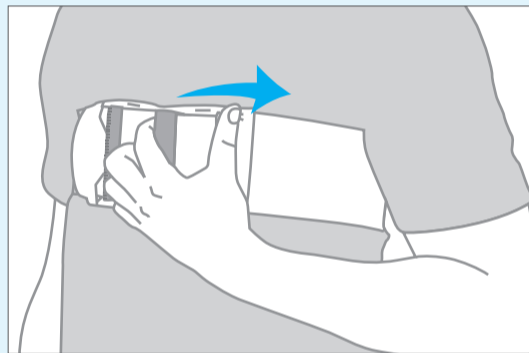

When you need to cough, slide the handle across your chest to tighten BracePlus™. You can now cough freely and securely.

To Mobilise... Pull & Lock

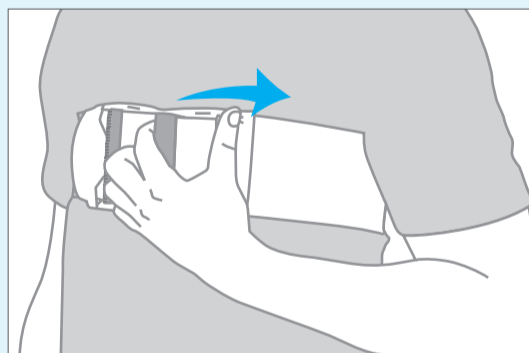

When you would like more freedom of movement, slide the handle across and use the brake to temporarily secure BracePlus™. Be careful not to press the release buckle when using the brake.

For maximum comfort wear BracePlus™ at all times apart from when washing.